

Concentrate

Bind-It™

Designed for Nuclear Medicine

Highly Effective for Tc99m,
Radioiodines and other
Nuclear Medicine Isotopes



237 ml

Bind-It™ Concentrate

Bind-It™ contains no harsh chemicals and is safe to use on delicate NaI(Tl) detectors. Bind-It™ traps and binds isotopes in solution, allowing them to be effectively removed from almost any surface. Bind-It™ will not cloud lead glass.

Bind-It™ is highly effective on Tc99m, I131, I125, I123 and nuclear medicine isotopes.

Directions for Use

Dilute with **distilled** water at a ratio of 10 parts water to 1 part Bind-It™ Concentrate. Thoroughly wet contaminated surface with Bind-It™ solution and let stand for 60 seconds. Wipe dry with a clean paper towel. Heavily contaminated areas may require repeated applications.

Cautions

Bind-It™ is for external use only. Avoid contact with sensitive areas of the body, eyes nose and mucus membranes.

IRRITANT
MAY IRRATE EYES
Do not get in eyes
Keep out of reach of children.

Contains amides, coco, N,N-bis(hydroxyethyl).

If swallowed, call a Poison Control Center or doctor immediately. Do not induce vomiting. If in eyes, rinse well with water for 20 minutes.

Keep away from children and pets.

Made in the USA


217 Dempsey Drive Elburn, IL 60119
www.labtechinc.com 1-800-542-1123