

Ready-To-Use Spray

Bind-ItTM

Radioactive Decontaminant

Highly Effective for Tc99m,
Radioiodines and other
Nuclear Medicine Isotopes



946ml

Bind-It™ Ready-to-Use Spray

For Effective Removal of Nuclear Medicine Isotopes

Purpose

Bind-It™ contains no harsh chemicals and is safe to use on delicate NaI(tl) detectors. Bind-It™ traps and binds isotopes in solution, allowing them to be effectively removed from almost any surface. Bind-It™ will not cloud lead glass.

Bind-It™ is highly effective on Tc99m, I131, I125, I123 and nuclear medicine isotopes

Directions for Use

Spray Bind-It™ decontamination fluid directly onto the surface to be cleaned. Thoroughly wet surface and let stand 30-60 seconds before wiping dry with a clean paper towel.

To clean electronics, first remove power. Thoroughly dampen a paper towel with Bind-It™, then gently scrub the surface.

Cautions

Bind-It™ is for external use only. Avoid contact with sensitive areas of the body, eyes nose and mucus membranes.

IRRITANT

MAY IRRATE EYES

Do not get in eyes

Keep out of reach of children.

Contains amides, coco, N,N-bis(hydroxyethyl).

If swallowed, call a Poison Control Center or doctor immediately. Do not induce vomiting. If in eyes, rinse well with water for 20 minutes.



Keep away from children and pets.

Made in the USA

217 Dempsey Drive Elburn, IL 60119 USA
www.labtechinc.com 800-542-1123