



Hand & Body Soap
TM
Bind-It
Designed for Nuclear Medicine

**Recommended for thyroid
patients receiving I131
treatment**

LTI LABORATORY
TECHNOLOGIES, INC.

237ml

Bind-It™ Hand & Body Soap

To Remove Nuclear Medicine Isotopes from Skin

Bind-It™ Hand & Body Soap is the most effective way to safely remove nuclear medicine isotopes from your skin and hair. Bind-It™ “traps and binds” the isotopes in solution allowing for their effective removal.

Bind-It™ Hand & Body Soap is an integral part of the Bind-It™ Patient Care Pack. The safest way to remove home contamination caused by I131 patients and minimize family exposure.

Directions for Use

Wet hands with water, pump Bind-It™ Hand & Body Soap into the palm of your hand. Rub skin vigorously for 60 seconds, thoroughly coating skin with soap. Rinse with plenty of clean water.

Cautions

Bind-It™ is for external use only. May be harmful if swallowed. Avoid contact with sensitive areas of the body, eyes, nose and mucous membranes.

In case of eye contact, flush with large amounts of water. If swallowed contact a Poison Control Center.

Keep away from children and pets.

Made in the USA

LTI LABORATORY
TECHNOLOGIES, INC.

217 Dempsey Drive Elburn, IL 60119
www.labtechinc.com 1-800-542-1123